

# Product information

## MD 535 plaster remover



### MD 535 at a glance

- Ready-to-use solution for the removal of plaster and alginates from impression trays, spatulas, plaster knives, mixing beakers (including vacuum mixing devices), prosthetic materials of all types, orthodontic appliances, etc.
- Excellent cleaning power - eliminates even the most stubborn deposits.
- Suitable for use in the ultrasonic cleaner.
- With activity indicator.

### Properties

MD 535 from Dürr System Hygiene is a special, high-performance cleaner for removing plaster and alginates from impression trays, mixing spatulas, plaster cutters, mixing bowls (including of vacuum mixers), all types of prosthetic materials, orthodontic regulators, etc.

### Product composition

MD 535 is based on a combination of complexing agents (15-30%), alkaline cleaning components (< 5 %) and excipients.

### Directions for use

MD 535 is applied undiluted. Immerse components to be cleaned in MD 535 and allow for the solution exposure time to elapse. After treatment, rinse the cleaned components with water. The cleaning process is accelerated using an ultrasonic cleaner and by heating the solution to a maximum of 50°C. Replace solution as needed. Rising gas bubbles indicate the cleaning action. If this no longer occurs, a fresh solution must be used. Residue in the solution does not impair the effect of MD 535.

### Environmental impact

The packaging is made of polyethylene and polypropylene and can thus be recycled or incinerated. For recycling, flush bottles with water.

For further information see safety data sheet.

### Shelf life

Do not use the product after the expiry date printed on the packaging.

### Container size

2.5-L bottle

### Storage

Store in a well-ventilated place. Keep container tightly closed.

### General notes

For chrome and nickel-plated impression trays, check compatibility with MD 535. Do not insert the aluminium impression tray in MD 535 for longer than 1 hour. Changes in product colour/odour may occur, especially when stored in sunlight. However, these changes in colour and/or odour have no impact on the cleaning efficacy of the product. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

# Product information

## MD 535 plaster remover

### Hazard warnings

MD 535 is classified and labelled according to the CLP Regulation. For further information see safety data sheet.

pH: 7.5–8.5



### Sales

Dürr Dental SE  
Höpfigheimer Str. 17  
74321 Bietigheim-Bissingen  
Germany  
[www.duerrdental.com](http://www.duerrdental.com)  
[info@duerrdental.com](mailto:info@duerrdental.com)

### Manufacturer

orochemie GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim  
Germany  
[www.orochemie.de](http://www.orochemie.de)  
[info@orochemie.com](mailto:info@orochemie.com)